"The Clinical Advantage"



Early rehabilitation, BWSTT or over ground...

Partial Body-Weight Support

Reduce patient body weight with single-point suspension for proper gait mechanics.

www.biodex.com/nxstep





NxStep Unweighing System Provides a safe rehabilitation environment for patient and therapist.

FEATURES:

- Full Patient Access open frame design encourages therapist facilitation and clears an unobstructed forward view for patients.
- **One-Handed Control** press a button to raise or lower patient; another to set unweighing load.
- **Counters Harness Slippage** Auto Unload feature compensates for potential harness movement due to slippage or posture changes.
- **Convenient Control** newly designed handheld remote can be magnetically attached anywhere on the frame.
- **Safe Environment** dynamic, single-point suspension is safer for patients and negates the need for therapist support and risk of personal injury.
- **Pelvic Stabilization** possible as single- or two-point stabilization with retention cords.
- Tall and Small from pediatric up to 6' 7" on a standard treadmill in an area with 8' ceilings or up to 7' 2" over ground.
- **Get Comfortable** therapist seat can easily be moved along the base of the frame or from side-to-side. A second seat is available separately.
- Handrails As Needed easily removable for specific balance training exercise.
- **Room-to-Room** collapses to 32" (81 cm) wide to fit through standard door frames.

Restore functional mobility

The Biodex NxStep[™] Unweighing System enables partial weight-bearing therapy with open access to the patient. Offloading a percentage of body weight enables the opportunity for early rehabilitation while providing a safe environment for patient and therapist. Therapists can focus on treating their patients, manually facilitating lower extremities.

The dynamic, single-point suspension of NxStep accommodates pelvic rotation and vertical displacement. This encouragement of proper mechanics allows functional gait patterns to be practiced. When used in combination with the Biodex Gait Trainer, a comprehensive gait training environment complete with audio and visual biofeedback, is possible.

Gait Kinematics

NxStep[™] Unweighing System allows full and partial weight bearing without compromising proper gait kinematics. Depending on walking speed, up to four inches of vertical displacement is permitted while the system maintains a consistent level of unweighing. When pelvic stabilization is necessary, retention cords are provided that can be adjusted for degree.



The ability to allow pelvic rotation has its own advantages. Unlike two-point suspension systems that have a tendency to restrict rotation on a horizontal plane, the single-point method of NxStep permits functional pelvic rotation and versatility when walking, side-stepping, retro-walking and turning. Patients can engage in reactive postural control exercises or change direction without repositioning the entire support system.

The unique, forward corner-mounted design of NxStep enables many advantages. Therapists can view patients from all angles; patients have full visibility of treadmill display and, the open design allows natural arm swing, an essential component to encourage neuroplasticity.



BIODEX

NxStep enables reactive, postural control exercise.

NEW STANDARD UNWEIGHING HARNESS

The question isn't whether to offload a patient's weight, but how. What factors to consider should be textbook, allowing the biomechanics of gait without impeding movement.

The Biodex Unweighing Harness is designed to provide vertical lift through three shared points of pressure: through the hip straps, gluteal fold straps and the lumbar-thoracic harness. Its secure fit with maximum support offers comfort without limiting lower extremity range of motion. Assuring that the patient is comfortable is essential when lengthy exercise sessions are required.

The NxStep Unweighing System allows the necessary movements to replicate natural ambulation – uniquely offered with Biodex design.

The new Auto Unload feature compensates for any harness slippage, maintaining unloaded amount and position throughout the exercise.

Take it a step further and partner the NxStep Unweighing System with the Biodex Gait Trainer. This combination provides the safety to treat people with movement disorders, and the biofeedback technology with objective reporting to prove need, progress and outcome.







Biodex NxStep Unweighing System shown with the Standard Unweighing Harness and two Therapist Seats.

Biodex NxStep Unweighing System shown with Gait Trainer.

NxStep allows functional gait patterns to be practiced.

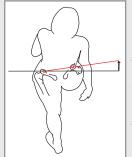

Pelvic rotation is a critical component of functional gait mechanics. As patients progress in therapy, the Biodex single-point suspension system allows functional pelvic rotation to occur.

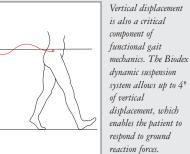

APPLICATIONS:

- Stroke
- Spinal Cord
- Head Injury
- Amputees
- Orthopedic
- Neurologic
- Vestibular
- Older Adult

SPECIFICATIONS:

Dimensions:

Operational: I.D.: 38.8" w x 48" d x 94" h (99 x 122 x 239 cm) O.D.: 47.5" w x 48" d x 94" h (121 x 122 x 239 cm) Retracted (not for operation) O.D.: 32" w x 48" d x 79" h (81 x 122 x 201 cm) will fit through a standard 36" x 80" door. Battery adds 4" (10.2 cm) to depth.

- Accommodates patients from pediatric up to 6' 7" (201 cm) on a standard treadmill in an area with 8' (244 cm) ceilings
- Includes one therapist seat
- Accommodates Treadmill up to 38" w x 13.5" h (97 x 34 cm)
- Vertical Adjustment: 50" (127 cm)
- Unloading Weight Capacity: 160 lb (73 kg)
- User Capacity: 400 lb (181 kg)
- Power: 115 VAC
- Battery: Rechargeable 24V. Battery automatically charges
 when system is plugged in.
- Weight: 275 lb (125 kg)
- Compliance: IEC 60601-1 and ISO 10535 for safety; EMC to 60601-1-2.
- CE conformity to IEC 60601-1
- Warranty: Two years parts; one year labor

CE

950-485 NxStep Unweighing System, 115 VAC Includes: Standard Unweighing Harness, one Therapist Seat and a set of Pelvic Stabilization Retention Cords

Export Models Available.

Optional 950-486 Therapist Seat


Replacement

945-462 Retention Cords, Pelvic Stabilization

Additional Harness options available. See <u>www.biodex.com/harness</u>



SIMPLE OPERATION

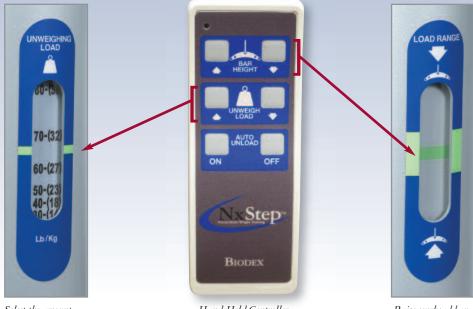

Select the amount of weight to offload the patient.

Hand-Held Controller

Raise overhead bar in Load Range until indicator is in the green zone.



A New Partial Body-Weight Support System that provides an open, accessible rehabilitation environment with automated harness compensation.

www.biodex.com/nxstep



20 Ramsey Road, Shirley, New York, 11967-4704, Tel: 800-224-6339 (Int'l 631-924-9000), Fax: 631-924-9338, Email: info@biodex.com, www.biodex.com